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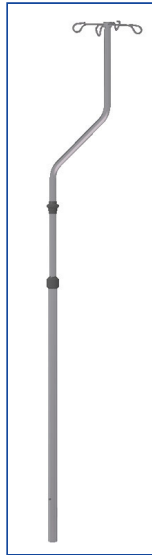
## Infusionsstativ

(Model ZP-3057/ZK-939 S/ZK-939 R/  
ZK-939U)

Nach EN 60601-1 

Directions for assembly and use

## IV-Pole

(Model ZP-3057/ZK-939 S/ZK-939 R/  
ZK-939U)

according to EN 60601-1 

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### 1. General / purpose:

- The above mentioned bended IV- Pole as well as a straight Iv-Pole should be used to be fixed at the bed and to carry gravity infusions.
- To ensure that the comprehensive safety demands on spaces used for medical purposes are met to your satisfaction, **the safety instructions have to be heeded.**

### 2. Safety instructions:

- It is absolutely necessary when setting up, handling and using the device that these instructions are followed precisely.
- The devices are only to be used for the purposes named in the directions for use. Any other use may conceal unknown hazards and is to be refrained from.
- The allowed maximum weight per bottle-hook is 2, 0 kg.
- Heavy objects like lead aprons etc. are not to be hung on the IV-Pole. The stand must be retracted before moving the bed to prevent collisions with doors or similar.
- The stand must be securely attached to the bed to prevent rotation in the bed retainer.
- Angled stands - Interior pipes may rotate in case of one-sided strain.
- Excess strain may impair the height adjustment function.
- Observe the safety instructions on the product. Observe the instructions in the bed's operating manuals.
- The stand may only be used in combination with the indicated type of bed.
- **Attention:** adjustment of the bed components may cause collisions with the stand or parts attached to it.
- Do not use excess force to pull the interior pipe upwards against the socket. This may damage the socket of the external pipe.

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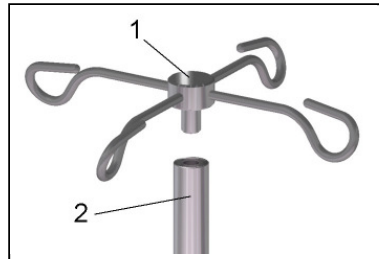
- The IV—Poles are NOT suitable for automated washing system. They should be cleaned by hand with a wet cloth.
- The person, who is selling the product, is only responsible for the safety and proper functioning of the IV-Poles when the directions for assembly and use have been followed in all points and only when original accessories are used.

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### 3. Assembling

#### Assembling the steel bottle holder:

- Screw the bottle hanger (1) into the  $\varnothing$  18 mm tube (2) with the M10 inner-nut and tighten.

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### 4. Cleaning and Disinfection:

The infusion stand can be treated with mild cleaning agents, for the disinfection you should use commercially available disinfection agents for example: Amocid, Baktolan, Sagrotan.

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### 5. Maintenance and Repair:

The infusion stand should be checked at least once a year for proper functionality of all important components:

Clamp-function for the inner tube  $\varnothing$ 18 still dequate.

Repairs have to be done by authorized personal or customer service with original equipment only.

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### 6 Ambient conditions:

For transport and stock-keeping up to 3 months, the following stock-keeping conditions have to be heeded:

Temperature: -25 bis 70 °C  
Moisture: 30-75%  
Atmospheric pressure: 700-1060 hPa

At application you have to assure the ambient temperature from 10 °C to 75 °C.