


Quick reference guide for healthcare bed model 3010


A. Side rails

To raise the side rails, pull the side rail upwards (1). Start with the part of the side rail on the headside of the bed.

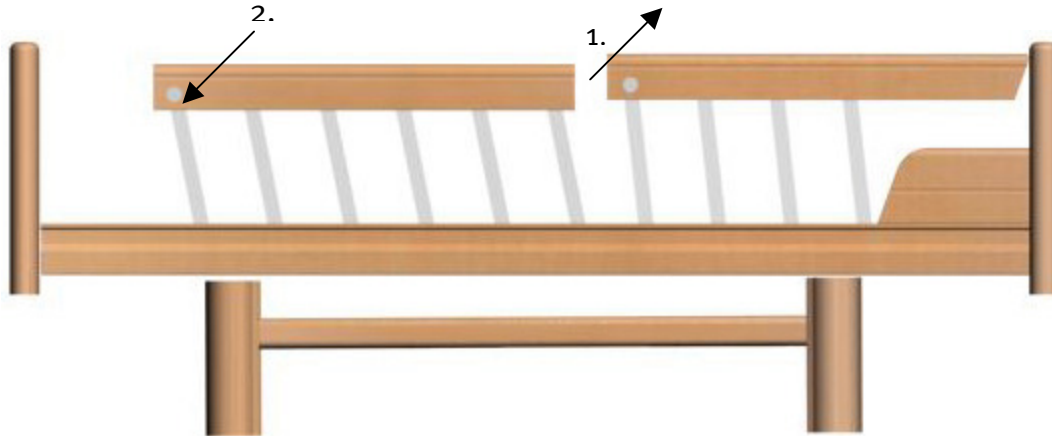

To lower the side rails, press the button at the lower part of the respective element (2) and move the side rail downwards. Start with the part of the side rail on the footside of the bed.

B. Manual control unit

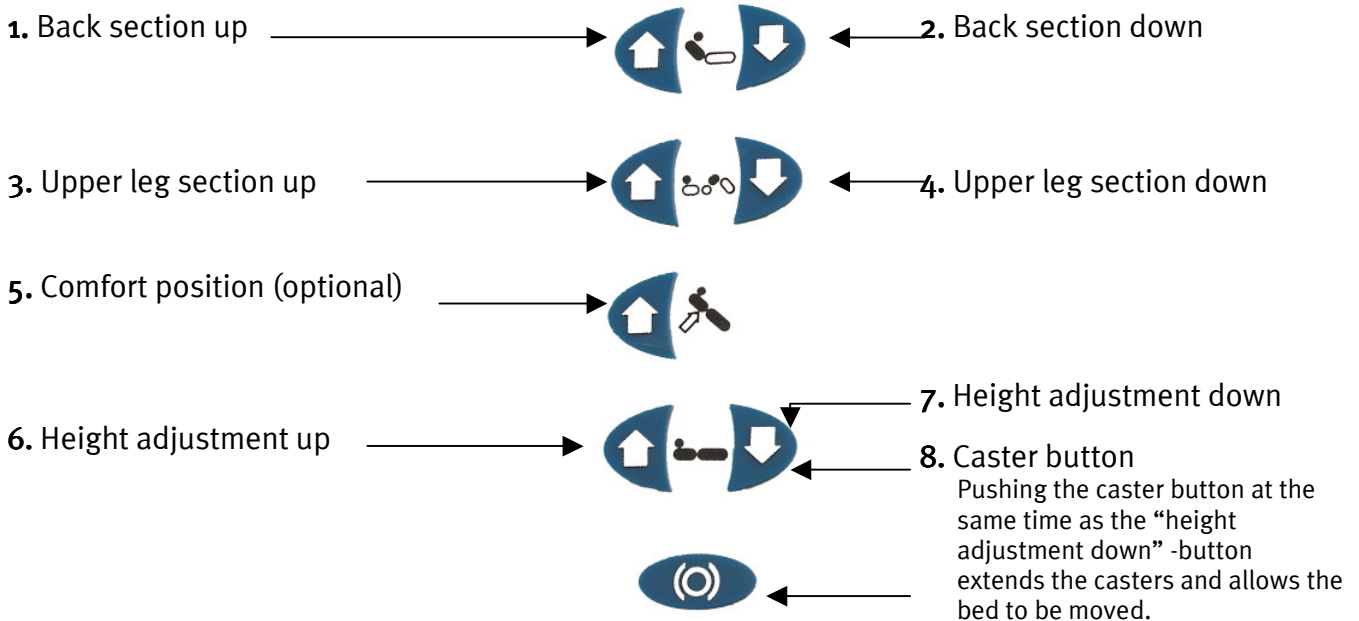

Quick reference guide for healthcare bed model 3010

C. Caregiver`s console (optional)

Trendelenburg


Tilt's the entire lying surface with the foot section high.

Comfort position

Tilt's the entire lying surface with the head section high.

Back section

Raises and lowers the back section.

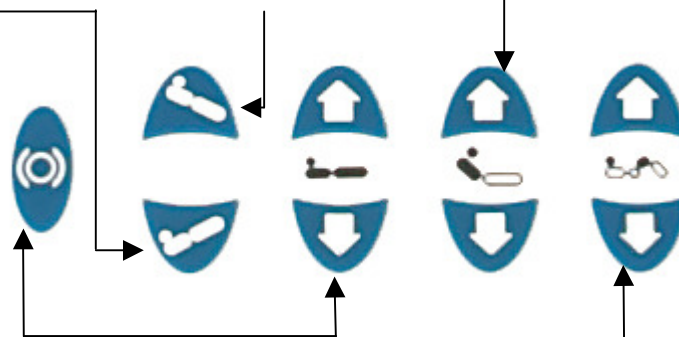

Caster button

Pushing the caster button at the same time as the "height adjustment down" -button extends the casters and allows the bed to be moved.

Height adjustment *

Raises and lowers the lying surface of the bed

Leg section

Raises and lowers the leg section.

D. Control lock

a) Manual control unit

By activating the control lock on the backside of the manual control unit, you lock all functions of the manual control unit.

b) Caregiver`s concole

By activating the control lock on the caregiver`s console, you lock all functions of the caregiver`s console and the manual control unit.



Activate the control lock



Deactivate the control lock



If the functions of the bed can not be operated, please check the control lock on the manual control unit and the caregiver`s console.



Please pay attention to the information in the operating instructions.