


A. Assist rails :

To raise the assist rails:

- Grasp the wooden assist rail top and slide from beneath the lying surface, then lift upward until it locks in place (be sure the assist rail is securely locked in the raised position).
- To further raise the assist rail, pull the wooden top upward.

To lower the assist rails:

- Push both buttons on the upper outside of the assist rail to lower.

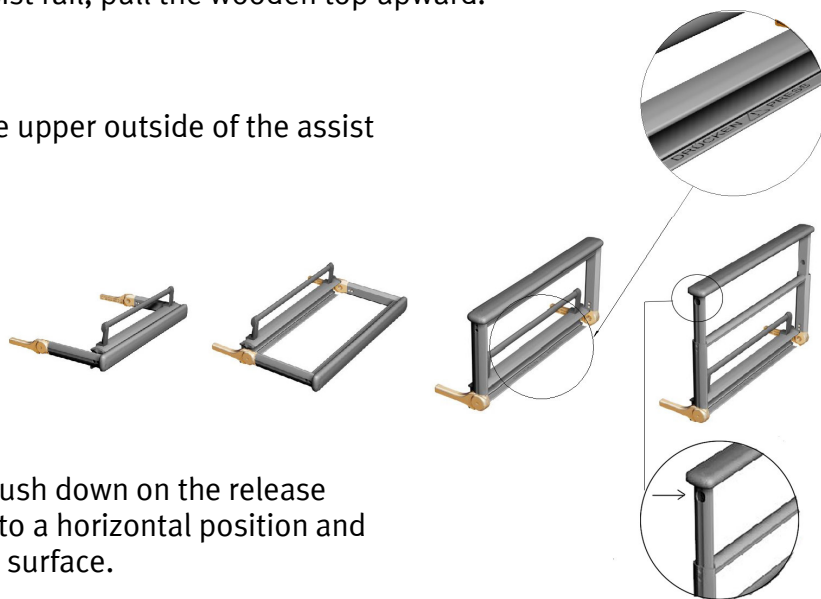

- To store the assist rail, push down on the release bar, lower the assist rail to a horizontal position and slide it beneath the lying surface.

B. Hand control unit

- | | | |
|--|--|--|
| <p>1. Lying surface
back section up</p> <p>2. Lying surface
upper leg section up</p> <p>3. Comfort positioning
Reverse trendelenburg with
entire lying surface tilting
head high.</p> <p>4. height adjustment up
Raises entire lying surface</p> | | <p>5. Lying surface
back section down</p> <p>6. Lying surface
upper leg section down</p> <p>7. height adjustment down
Lowers the entire lying
surface.</p> <p>8. Unlocking the casters
Pushing buttons 7 and 8 after
fully lowering the bed
extends the casters.</p> |
|--|--|--|

C. Carergiver's keyboard

1. Height adjustment: „UP/DOWN“ :

Double-click-function up/down -
Allows continuous up or down
Movement of the lying surface with
Instant stop by pushing any button.

2. Hand control lock out

Locks all hand control and
console unit functions –
except Trendelenburg.

3. Trendelenburg-Position

Tilt's the entire lying surface with the foot section high.

4. Reverse-Trendelenburg-Position

Tilt's the entire lying surface
with the head section high.

5. lying surface : back section


Raises and lowers
the back section of
the lying surface.

6. lying surface : leg section


Raises and lowers
the leg section of
the lying surface.

7. Caster button

Pushing the caster button at
the same time as the height
adjustment downbutton –
extends the casters and
allows the bed to be moved.

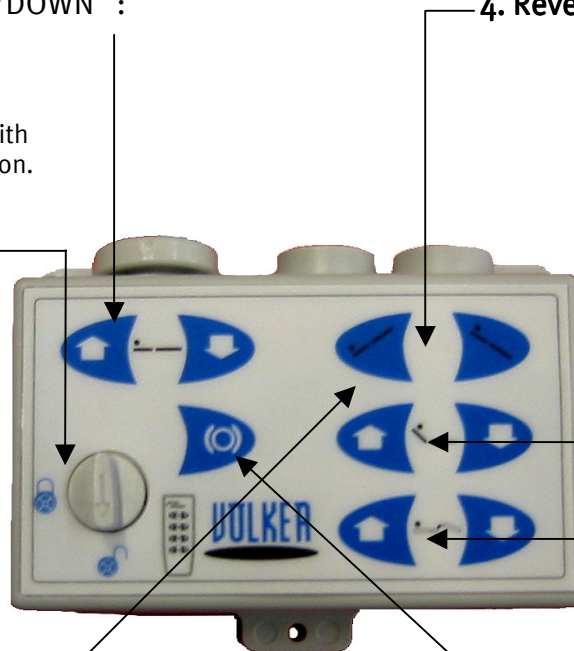

D. Raised foot section adjustment

Pushing button 2  moves the bed to the „raised knee“-position.

The foot section can then be raised manually.



Review the complete
operation manual before
operating the bed.