

Brief instructions for use

Models 3082 Vis-a-Vis, 3082 K Vis-a-Vis from year of construction 06/2009

VÖLKER


A. Side rails

Raising the side rails


1. Pull the side rail element out horizontally until the end stop, and fold it upwards.
2. To adjust the height of the side rail, pull the telescopic section upwards until it reaches its end stop.

Lowering the side rails

1. Press both buttons on the outside of the frame (fig. 1), just under the cross-member, to bring the height-adjustable rail element to its lowest position.
2. Press the "Drücken / Press" trigger at the lower edge of the side rail element (fig. 2) and tilt it sideways into the horizontal plane, so that it lies parallel to the floor.
3. Push the rail element completely underneath the lying surface (fig. 3a, fig. 3b).

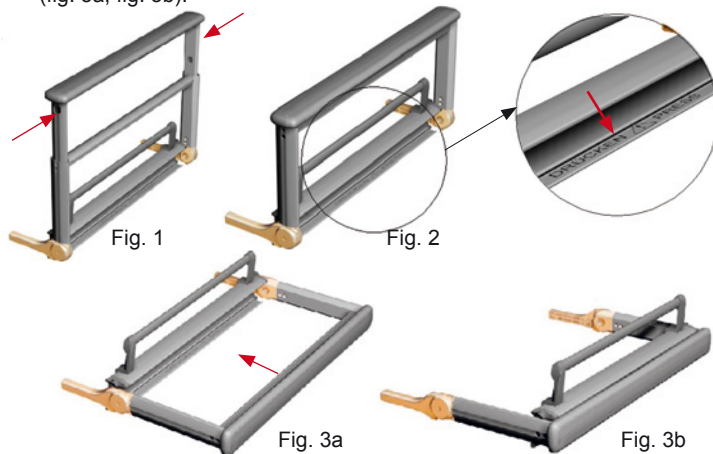

B. Hand control

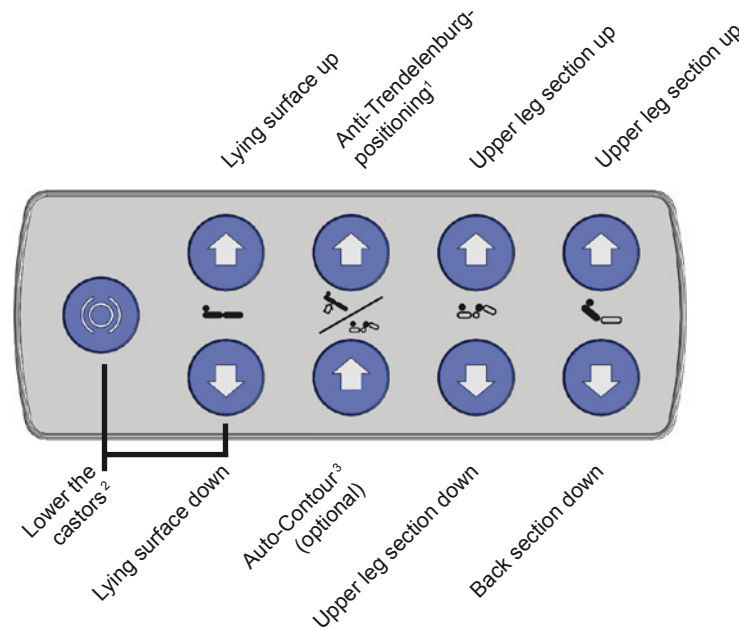

¹ Head end raised

² Press both buttons simultaneously; the lying surface moves to the lowest position and the castors lower


³ Back and upper leg section raised simultaneously

Brief instructions for use

Models 3082 Vis-a-Vis, 3082 K Vis-a-Vis from year of construction 06/2009

VÖLKER

C. Nurse hand control

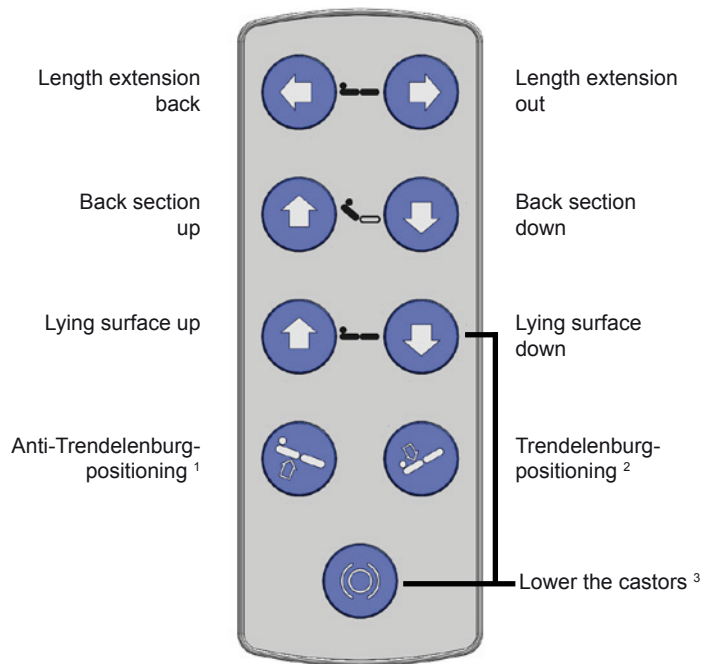

¹ Head end raised

² Head end lowered

³ Press both buttons simultaneously: the lying surface moves to the lowest position and the castors lower

D. Key lock

Activating the key lock of the hand control or the nurse hand control disables all of the functions of that control.



Hand control/
nurse hand control
locked



Hand control/
nurse hand control
unlocked

NOTE If no electric function can be activated, press the green reset button on the dual drive. Then press the green button on the mains isolation after about 10 seconds, and then confirm any hand control function. Never press both green buttons at the same time.

E. Vis-a-Vis position

NOTE It is only possible to extend the length when the bed is level (apart from back section) and in its lowest position with the locking mechanism closed.

If the lying surface is moved to its end point with the function key "Length extension out", the lower leg section can be moved sideways to the left or right after unlocking. The height adjustment of the lying surface (lifting mechanism) is locked in this position - the Anti-Trendelenburg position and the adjustment of the back and upper leg sections are possible.

NOTE The bed is not suitable for cleaning in automatic washing systems (IPX4).

NOTE Please observe the notes in the detailed instructions for use!