

# Brief instructions for use

**Model 5380, 5380 K** from year of construction 12/2011

**VÖLKER**


## A. Side rails

### Raising the side rails


1. Pull the side rail element out horizontally until the end stop, and raise it upwards.
2. To adjust the height of the side rail, pull the telescopic section upwards until it reaches its end stop.

### Lowering the side rails

1. Press both buttons on the outside of the frame (fig. 1), just under the cross-member, to bring the height-adjustable side rail element to its lowest position.
2. Press the "Drücken / Press" trigger at the lower edge of the side rail element (fig. 2) and tilt it sideways into the horizontal plane, so that it lies parallel to the floor.
3. Push the rail element completely underneath the lying surface (fig. 3a, fig. 3b).

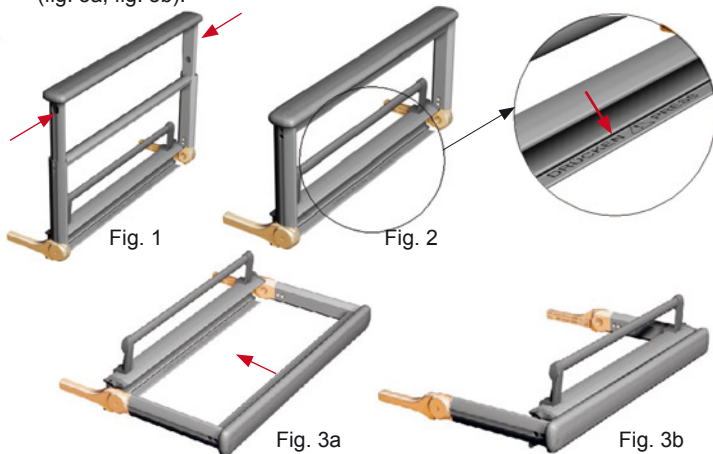

## B. Hand control

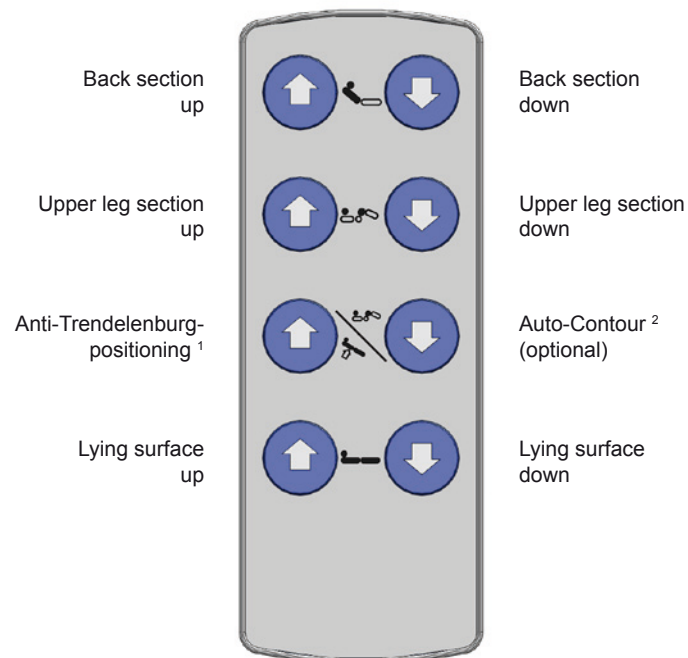

<sup>1</sup> Head end raised


<sup>2</sup> Back and upper leg section raised simultaneously

# Brief instructions for use

**Model 5380, 5380 K** from year of construction 12/2011

**VÖLKER**

## C. Nurse keypad with individual lock (option)

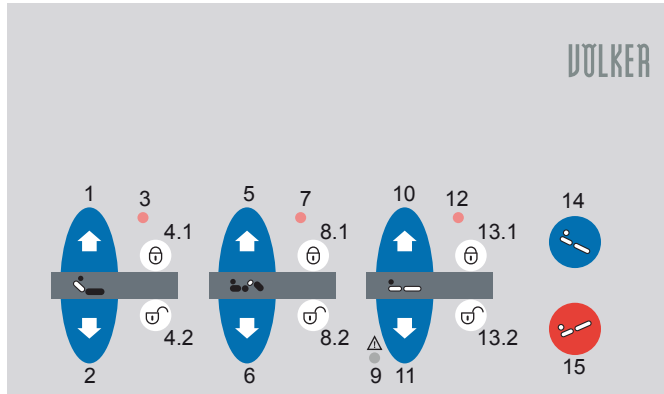

- |                                   |  |
|-----------------------------------|--|
| 1 Back section up                 | 9 LED - Lowermost position of the lying surface adjustment |
| 2 Back section down               | 10 Lying surface up <sup>1</sup>                           |
| 3 LED - Back section locking      | 11 Lying surface down <sup>1</sup>                         |
| 4.1 Lock back section             | 12 LED - Height adjustment of the lying surface locked     |
| 4.2 Unlock back section           | 13.1 Lock lying surface adjustment                         |
| 5 Upper leg section up            | 13.2 Unlock lying surface adjustment                       |
| 6 Upper leg section down          | 14 Anti-Trendelenburg positioning <sup>2</sup>             |
| 7 LED - Upper leg section locking | 15 Trendelenburg positioning <sup>3</sup>                  |
| 8.1 Lock upper leg section        |  |
| 8.2 Unlock upper leg section      |  |

<sup>1</sup> automatic function with double click possible on a country-by-country basis

<sup>2</sup> Head end raised

<sup>3</sup> Head end lowered

## D. Key lock

Activating the hand control key lock disables all of the functions of the hand control.



Hand control locked



Hand control unlocked

Locking a function on the nurse keypad also locks this function on the hand control.

## E. Castor fixing

Two of the four guide castors can be braked.

Foot pedal down: The bed is fixed.

Foot pedal up: The bed is ready for moving.

Both castors must be fixed if the bed will not move.

In the optional, manoeuvrable variant, the bed has four additional guide castors at the head end, allowing the bed to be manoeuvred on the spot. Two of these castors are provided with direction fixers. These must also always be fixed, apart from when moving the bed. The direction fixers are not correctly fixed until the foot pedals are down and the castors can be heard latching into place parallel to the length side of the bed after turning slightly.

**NOTE** The bed is not suitable for cleaning in automatic washing systems (IPX4) and may not be used for transporting occupants/patients.

**NOTE** Please observe the notes in the detailed instructions for use!