

# Additional information to the instructions for use

## Protection class of UK power plugs for Völker beds

### **NOTE**

- The UK power plug is designed as a plug for protection class I (separated earthing pin), but the earth protection pin is not connected to any earth wire as the medical bed is designed as a class II product.
- The design of UK power sockets requires an earth pin to be present on the plug even if the product does not require an earth connection.
- This design does not have any influence to the use of the product.
- Please be aware of the Class I power plug when making electrical safety measurements. The medical bed should be tested against Class II requirements.