


# Brief instructions for use

## Server RT-985, Server Vis-a-Vis

VÖLKER

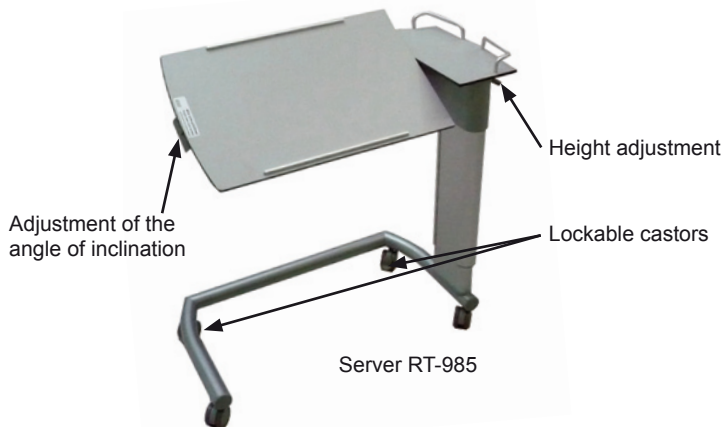

### Height adjustment

To bring the tray upwards, pull it up slowly. To bring it downwards, push the handle upwards and let the tray down carefully.



### Adjusting the angle of inclination

To adjust the angle of inclination of the tray, push the handle upwards and tip the tray in the desired direction.



**CAUTION** Do not lean on the server!



**CAUTION** The server is not suitable for transporting persons!



**CAUTION** Secure the castors of the server to make sure that it cannot roll away!



**CAUTION** There must be no objects on the tray while you adjust the angle of inclination of the tray!

**NOTE** The total weight of the objects on the server may not exceed 15 kg!